

Vertys Mineral


Composito fluido e in pasta fotopolimerizzabile per Corone e Ponti, Inlays e Faccette
Light cured flowable and paste composite for Crowns & Bridges, Inlays and Veneers



RIFERIMENTO CROMATICO

CHROMATIC REFERENCE

DENTINE FLOW *DENTIN FLOW*

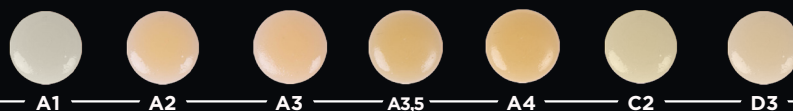


DENTINE PASTA *DENTIN PASTE*



SMALTI FLOW *ENAMEL FLOW*




SMALTI PASTA *ENAMEL PASTE*

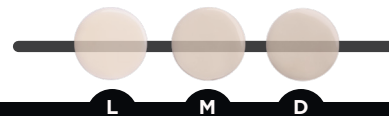


TRASPARENTI FLOW *TRANSPARENT FLOW*




TRASPARENTI PASTA *TRANSPARENT PASTE*

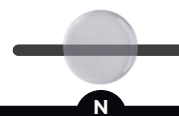


OPALESCENTI FLOW *OPALESCENT FLOW*

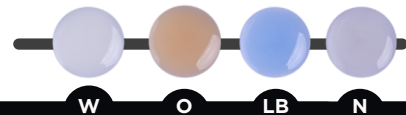


OPALESCENTI PASTA *OPALESCENT PASTE*

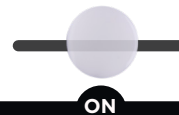


INTENSIVI FLOW *SHINE FLOW*

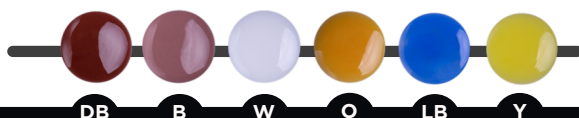

ADDITIVI FLOW *ADDITIVES FLOW*

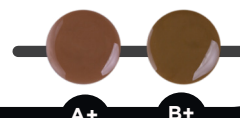

ROSA FLOW *PINK FLOW*

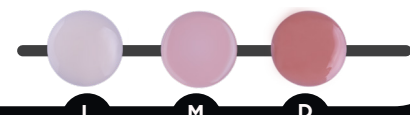

ROSA PASTA *PINK PASTE*






SUPERCOLORI *SUPERCOLORS*



TECNICA DI STRATIFICAZIONE METAL-FREE

METAL-FREE LAYERING TECHNIQUE




- Trasparente
Transparent
- Smalto
Enamel
- Dentina
Dentin
- Opaco
Opak
- Metallo
Metal

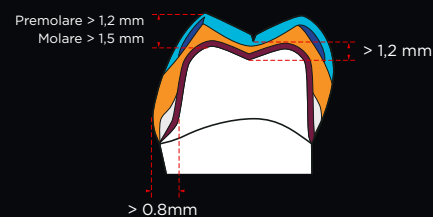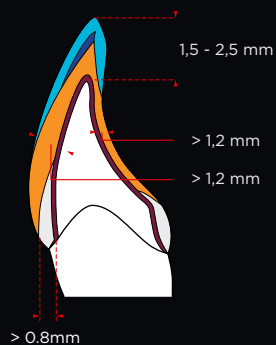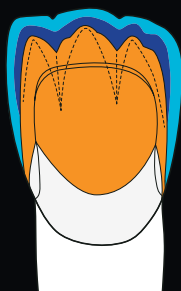

TECNICA DI STRATIFICAZIONE SU METALLO


LAYERING TECHNIQUE ON METAL

- Trasparente
Transparent
- Smalto
Enamel
- Dentina
Dentin
- Opaco
Opak
- Metallo
Metal

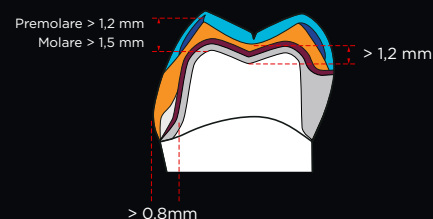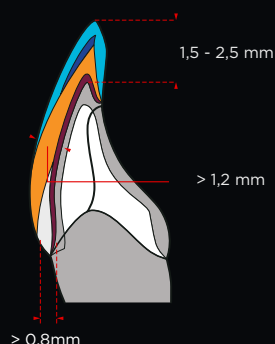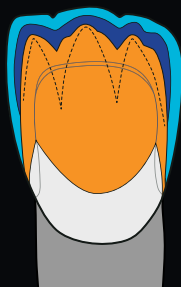

STRATIFICAZIONE INTERNA




INTERNAL LAYERING

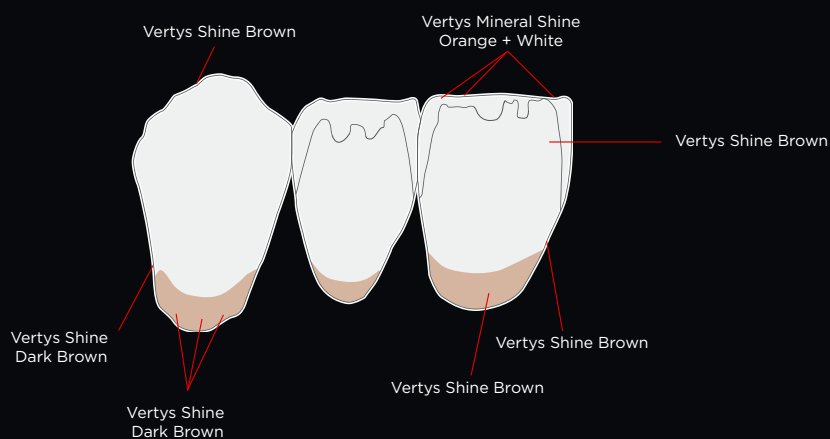

STRATIFICAZIONE ROSA

PINK LAYERING

- Trasparente
Transparent
- Smalto
Enamel
- Dentina
Dentin

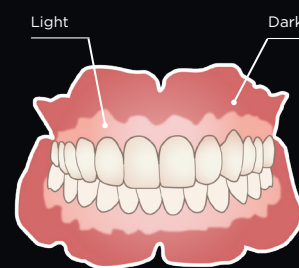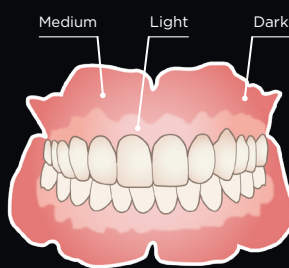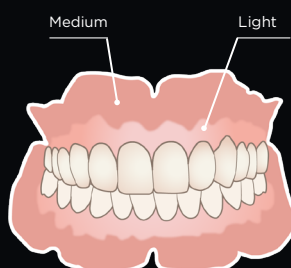

ABBINAMENTO DENTINA / OPACO / SMALTO

OPAK / DENTIN / ENAMEL COMBINATION

DENTINA DENTIN		OPACO OPAK	SMALTO ENAMEL
A1	A2	LIGHT	LIGHT
A3	A3,5	MEDIUM	MEDIUM
A4		DARK	DARK
C2	D3	MEDIUM	MEDIUM

TEMPI DI FOTOPOLIMERIZZAZIONE

LIGHT CURING TIME TABLE

MINERAL	STEP 1	STEP 2	FINAL STEP WITH MINERAL GEL
OPACO OPAK	1 min.	5 min.	
DENTINA DENTIN	1 min.	3 min.	5 min.
SMALTO ENAMEL	1 min.	3 min.	5 min.
BOND	10 s		
GLAZE	5 min.		
SUPERCOLORI SUPERCOLORS	20 s		



MOON LIGHT 2

I DATI SI RIFERISCONO AL FOTOPOLIMERIZZATORE
MOON LIGHT 2 / ALL DATA REFERS TO MOON LIGHT 2




System	Microprocessor
Source	365 nm and 485 nm UV LED
Display	3.2 inch Touch Screen Display
Programs	CLEAR, BASIC + 7 Customizable
Power input	Programs
LED power	100-240 V, 1A, 50-60 Hz 150 W

IMPOSTAZIONI SETTINGS




STRUTTURA GENGIVA

GUM STRUCTURE REFERENCE

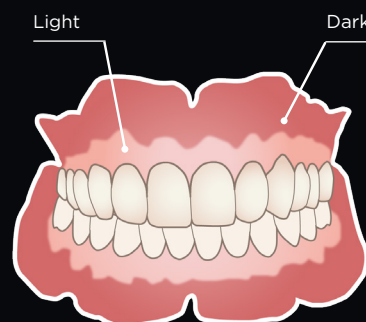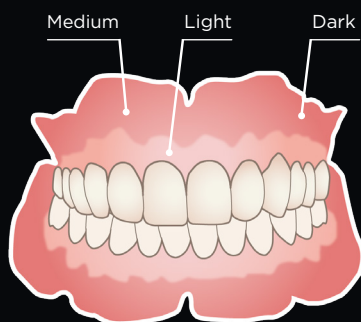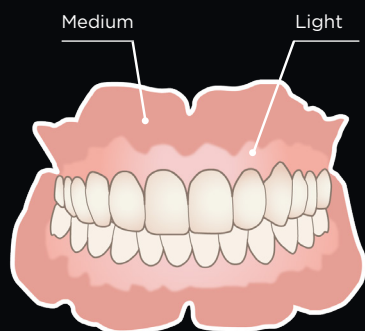

ROSA FLOW PINK FLOW

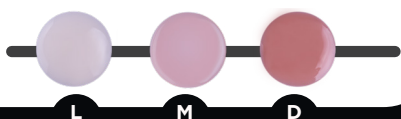

ROSA PASTA PINK PASTE



ROSA FLOW

PINK FLOW

200-3729	VERTYS MINERAL PINK FLOW LIGHT
200-3730	VERTYS MINERAL PINK FLOW MEDIUM
200-3731	VERTYS MINERAL PINK FLOW DARK

ROSA PASTA

PINK PASTE

200-3729	VERTYS MINERAL PINK PASTE LIGHT
200-3731	VERTYS MINERAL PINK PASTE DARK

PASSAGGI LAVORAZIONE GENGIVA

GUM PROCESSING STEPS

1. IRRUVIDIRE LA FLANGIA VESTIBOLARE DELLA PROTESI
2. APPLICARE IL BOND LIQUID E FOTOPOLIMERIZZARE PER 10 s
3. APPLICARE I COMPOSITI GUM COME INDICAZIONE
4. RIFINIRE LO SCUDO VESTIBOLARE
5. FOTOPOLIMERIZZAZIONE FINALE CON MINERAL GEL
6. LUCIDATURA FINALE

1. ROUGHEN THE BUCCAL FLANGE OF THE PROSTHESIS
2. APPLY THE BOND LIQUID AND LIGHT CURE FOR 10 s
3. APPLY GUM COMPOSITES AS AN INDICATION
4. FINISH THE VESTIBULAR SHIELD
5. FINAL PHOTOPOLYMERIZATION WITH MINERAL GEL
6. FINAL POLISHING

MATERIALI CONSIGLIATI

RECOMMENDED MATERIALS

200-3805	VERTYS MINERAL MODEL LIQUID
200-3050	ERGO BRUSH C.0
200-3051	ERGO BRUSH C.1

